





Swab Collection Kit User Instruction Manual

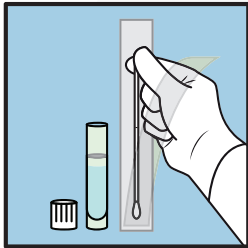


1 Open the package containing the swab, tube, and cap. Peel open the swab package and remove the swab.

Dip a fresh swab in a moistening buffer (Pink Saline tube provided with kit). With THUMB AND INDEX FINGER OF ONE HAND TO GENTLY GRASP BOTTOM OF EAR, PULL DOWN AND FORWARD USING THUMB AND INDEX FINGER. With the other hand, hold the swab so the shaft is parallel to THE SKIN BEHIND THE BOTTOM OF THE EAR ON THE SCALPSIDE BUT NOT ON HAIR. Rub the swab vigorously along the exposed ear fold back and forth 10 times (30 seconds)
NOTE: IT MAY BE EASIER TO HAVE SOMEONE ELSE DO THE SWABBING FOR YOU!

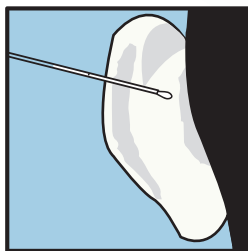


2

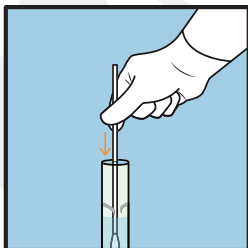


3

Holding the tube upright, insert the swab into tube and apply downward pressure to pierce the foil safety seal. The swab should be submerged into the reagent after piercing the seal.

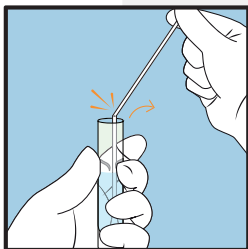


4

Break off the top of the swab at the breakpoint, leaving the lower section of the swab in the tube.

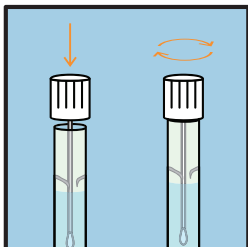


5

Attach the cap to the top of the tube. Tighten cap until completely closed.

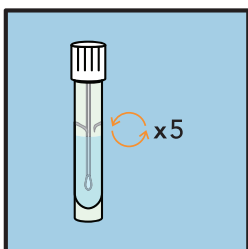

6

Invert the tube 5x. The sample is now stabilized and ready for transport

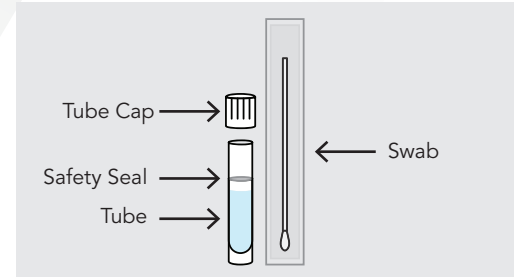

Complete the Chain of Custody (CoC) form supplied in the kit (also available online printed on our website)..

Return the sample with the Chain of Custody (CoC) document completed.

Reminders:

* Samples must be paid online OR by phone before processing the sample.

Ship to 11550 IH-10 West, Suite 105
San Antonio, Texas 78230

Contact Us:



support@envirobiomics.com

